CatchSystem CS

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Directions for use: GENERAL USE INFORMATION: The Catchsystem is intended for the purpose of harvesting bone and/or bone marrow specimens. WARNINGS AND PRECAUTIONS: This device is designed to be used by a physician. These instructions are not meant to define or suggest any medical or surgical technique. The Individual practitioner is responsible for the proper procedure and techniques to be used with this device. For use only for biopsies of bone/bone marrow as determined by a licensed physician. This needle should be used by a physician familiar with the possible side effects, typical findings, limitations, Indications and contraindications of bone biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anticoagulant medications. TAKESYSTEM PROCEDURE: 1. Using sterile technique, prepare the skin with antiseptic and drape. 2. Infiltrate the marked area with local anesthetic, especially the periosteum. 3. Make a skin incision with a scalpel blade over the marked area. 4. Using gentle, but firm pressure, advance the needle. Rotate the needle in an alternating clockwise-counter clockwise motion. Entrance into the marrow cavity is generally detected by decreased resistance. 5. Remove stylet. 6. Slowly and gently advance the needle a millimetre at a time, with clockwise -counter clockwise motion (for better cutting) until adequate marrow is obtained. 7. Insert the probe into the cutting cannula to check the sample length in the needle lumen. 8. Remove the probe and Insert the CatchSystem fully into the needle cannula. 9. Slowly rotate and remove the CatchSystem and cannula together. 10. Remove the extraction cannula from the needle cannula and push the specimen out with the probe. NOTE: For safety and convenience, we introduced the Anti-Stick. Probe Guide as a standard feature of the "J" type Bone Marrow Needle. The purpose of this device is two-fold: provide an easier method to align the probe in the needle tip for sample expulsion and protect the sharp tip to eliminate any accidental injury during sample expulsion. Check If the Inner package is unopened and damaged. In case of damaged Inner package, do not use the product. Check the expiry date and the gauge. Possible allergic reactions should be considered. After use consider it as waste material. Store in a cool and dry place. Protect from light. Use of the device is restricted to physician. Polyethylene Oxidesterilized. Sterility and integrity guaranteed only if observed the prescribed conditions. It must be used only in hospitals. Sterile - Non pyrogenic - Disposable Sterile if unopened and undamaged inner packaged.

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