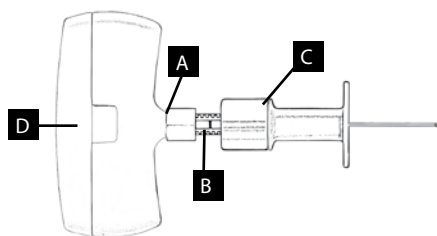
# Selective "ERG" Type - ITA

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Bone marrow aspiration device (ITA). Non-pyrogenic. Single use. Do not use this device in case of open or damaged package. Do not expose to high temperature and avoid light exposure.

## PRECAUTIONS

The procedure is considered only as a help guide. Each physician should evaluate the procedure here described according to its experience and type of need. Is a single use device. Zamar disclaims any liability for reuse of this device.

## DIRECTIONS FOR BONE MARROW ASPIRATION

1. The needle protection and set the cannula length by turning the ring "A". Needle length scale is shown in scale "B" in millimeters.
2. Check stylet mobility removing and lifting the cap "D" and fixing it back in the housing.
3. Perform local anesthesia, if required.
4. Firmly grip the needle with the palm and place it on patient's sternum or iliac crest. Push through the skin and keep on pushing, twisting left and right, till bone marrow cavity is reached at the pre-set needle length.

5. Extract and remove the cap of the stylet "D". Connect a syringe to the needle luer and aspirate.

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