
Ultimate PM-PMO

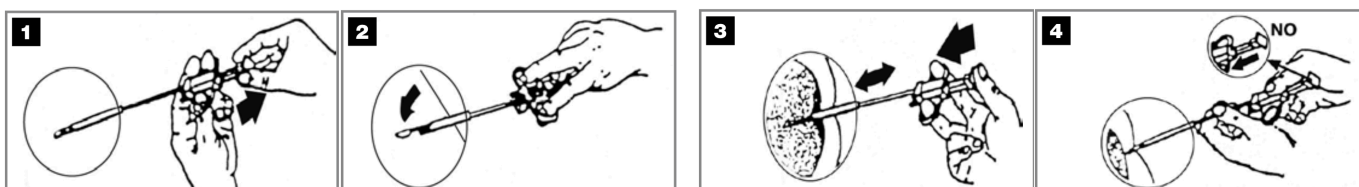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Directions for use: Soft tissue semi-automatic biopsy device. Non pyrogenic. Single use. Do not use this device in case of open or damaged package. Do not expose to high temperature and avoid light exposure. **PRECAUTIONS:** The procedure is considered only as a help guide. Each physician should evaluate the procedure here described according to its experience and type of need. Is a single use device. Zamar disclaims any liability for reuse of this device. 1. Load PM/PMO biopsy needle by pulling back hard on the plunger until a firm click is felt. 2. With stylet fully retracted (specimen notch completely covered by cannula as in fig 2) insert needle proximal to area to be sampled. Warning: be careful not to press plunger until needle is inserted into the lesion to biopsy. 3. Move plunger with thumb to first stop to advance the stylet and expose the specimen notch. Press the plunger past stop with thumb, automatically triggering the cannula to close, trapping a specimen in the sample notch. 4. Draw needle out. Pull back on plunger to recock the spring. Push plunger to first stop exposing the specimen from notch. Remove tissue specimen from notch in stylet. Be careful not to push plunger after first stop during this step.

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