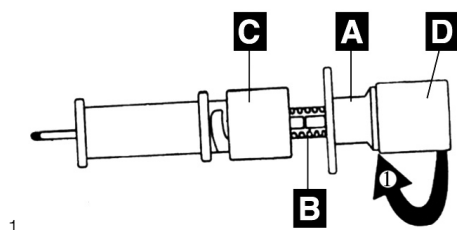
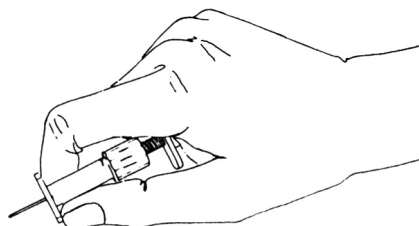
Selective-ST Selective E-STE

1.

2.

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Directions for use: Bone marrow aspiration device (ST) and bone marrow explantation device (STE). Non-pyrogenic. Single use. Do not use this device in case of open or damaged package. Do not expose to high temperature and avoid light exposure. **PRECAUTIONS:** The procedure is considered only as a help guide. Each physician should evaluate the procedure here described according to its experience and type of need. Is a single use device. Zamar disclaims any liability for reuse of this device. **DIRECTIONS FOR BONE MARROW ASPIRATION:** 1.Fig.1.The needle protection and set the cannula length by turning the ring "C". Needle length scale is shown in scale "B" in millimeters. 2.Check stylet mobility unscrewing and lifting cap "D" and then screw it back in the housing. 3.Perform local anesthesia, if required. 4.Grip the needle as described in Picture 2 and place it on patient's sternum or iliac crest. Push through the skin and keep on pushing, twisting left and right, till bone marrow cavity is reached at the pre-set needle length. 5.Unscrew the cap "D" and remove completely the stylet. Connect a syringe to the needle luer and aspirate. **DIRECTIONS FOR BONE MARROW: EXPLANTATION:** The procedure is similar to the bone marrow aspiration, follow the same procedure described in bone marrow aspiration by the point 1 to point 5.

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