## TPS

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Thoracentesis closed circuit set. Pyrogen free. Do not use the device if the packaging is open or damaged. Avoid high temperature and under light storage.

## PRECAUTIONS

La Zamar si esime da ogni responsabilità in caso di riutilizzo. The following procedure is only an helping guide the operator should use his clinical experience to perform the procedure. The set is a single use. Zamar is not responsible of the misuse.

## CONTENT

3 needles of 14G,16G and 18G drainage bag of 2.000 ml with anti-reflux valve and bottom tap 3 way stopcock 60 ml LL sy-

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ringe extension line.

## USE AND INSTRUCTIONS

1. Connect bag, extension line, 3-way stopcock, syringe and the requested needle together (the needle must be the requested diameter among the available).
2. Skin local anesthesia where request.
3. Enter the skin with, the needle, few centimeters lower the liquid to be drained out the lung.
4. Set the stopcock in a position where the syringe suction is possible for the diagnostic sampling, use the syringe to drain out the liquid by the pleura cavity and then set the stopcock connecting syringe

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and drainage bag to empty the syringe. Repeat the procedure if requested.

5. Before to start the procedure, to fix the needle avoiding unexpected misplacement is suggested.