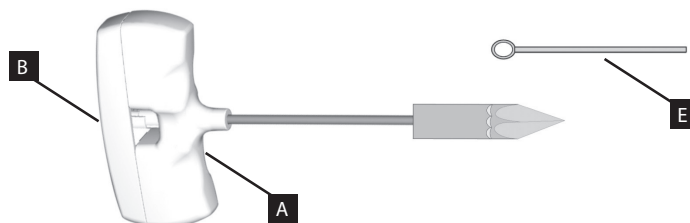
# Cortex - BP

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Bone marrow biopsy device (BP). Non-pyrogenic. Single use. Do not use this device in case of open or damaged package. Do not expose to high temperature and avoid light exposure.

### PRECAUTIONS

The procedure is considered only as a help guide. Each physician should evaluate the procedure here described according to its experience and type of need. Is a single use device. Zamar disclaims any liability for reuse of this device.

### BONE MARROW BIOPSY PROCEDURE

1. Lay the patient on the left or right lateral side on the operation table;
2. Find the iliac crest;
3. Make a local anesthesia and in particular on (periosteal search on google che io non riesco);

4. Cut 3 mm of patient skin
5. Firmly hold the needle handle and penetrate till the iliac crest;
6. With a light pressure moving the needle clockwise and anti-clockwise penetrate (compatta guarda su google come si traduce che io non ho il computer, scusami);
7. You can easily feel when the needle reach the bone cavity because it needs a very soft pressure to enter!
8. Pull the inner stylet out moving the cap "A" anti-clockwise;
9. Place the cap "B" on the needle handle to avoid blood leaking
10. Penetrate the bone marrow moving the needle in one direction only, once you have reached the right needle tip position, simply pull the needle out because the special tip conformation will hold a bone marrow core inside. To help you operating on harder tissue you could enter bone marrow and make a tip deflection to detach the bone marrow core!the deflection procedure helps to get the bone

marrow sample inside the needle tip. To release the bone marrow sample out of the needle, remove the "B" cover and insert in the needle the stylet "E" pushing it inside till you have got the bone marrow sample out!

NOTE: to make bone marrow aspiration connect a luerlock syringe to the needle once the needle is already insert in the bone marrow cavity and aspirate before to make the bone marrow biopsy.

WARNING: Don't move lateral the needle without stylet inside the trabecular bone.

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